EN

FR

DE

JA

ZH

KO

ShockLink

Important Product Information
重要产品信息

English

ShockLink Important Information

Read these instructions thoroughly. Observe all warnings, precautions and instructions on the product, in the User Guide and in this Important Product Information booklet. Retain this booklet for future reference.

Warnings and Cautions

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the product.

Note

A note states important information about the product or its operation.

Cautions

- Maximum energy in one shock: 360 J (Only give a cumulative total of 1,000 J per minute.) ShockLink will disconnect the defibrillator for one minute if a cumulative total of more than 4000 J has been given over the last 3 minutes
- Do not use ShockLink if the cable or the housing has visible damage or openings.
- Do not use ShockLink in wet conditions.

Waste Handling

Dispose of in accordance with your country's recommendations.

This appliance is marked according to the European directive 2012/19/EC on Waste Electrical and Electronic Equipment (WEEE).

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

The symbol on the product, or on the documents accompanying the product, indicates that this appliance may not be treated as household waste. Instead it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment. Disposal must be carried out in accordance with local environmental regulations for waste disposal.

For more detailed information about treatment, recovery and recycling of this product, please contact your local city office, your household waste disposal service or Laerdal representative.

Federal Communications Commission (FCC) and Industry Canada (IC) Statements

This device complies with part 15 of the FCC Rules and RSS-210 of IC rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Changes or modifications not expressly approved by Laerdal Medical could void the user's authority to operate the equipment.

The term "IC" before the equipment certification number only signifies that the Industry Canada technical specifications were met.

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This device contains licence-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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MSIP-CMM-Lm1-185-20050

EU

CE: This product is in compliance with the essential requirements of Council Directive 2014/53/EU on Radio Equipment (RED), and Council Directive 2011/65/EU on Restriction on the use of certain hazardous substance (RoHS).

English

ShockLink Important Information

UK

This product complies with the requirements of U.K. legislation of Radio Equipment Regulations (S.I. 2017/1206).

UKCA (U.K. Conformity Assessed): This product complies with the requirements of U.K. legislation of Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations (RoHS) Regulations (S.I. 2012/3032).

Symbol Glossary

	CE Mark
	UKCA Mark
	Australian Radiocommunications and EMC Compliance Mark
	Korean Certification (KC) Mark
	Dispose of in accordance with your country's recommendations
	FCC Statement and IC statements
	Reference order number
	Production lot number

Specifications

Dimensions	85 × 53 × 14 mm (3.3 × 2 × 0.5")
Weight	120 g (0.26 lbs) (including battery)
Cable length	Approximately 0.9 m (3ft)
Defibrillator impedance	115 Ohms
Operation temperature	0 °C to +35 °C (32 °F to 95 °F)
Storage temperature	-15 °C to +50 °C (5 °F to 122 °F)
Humidity	< 95% relative humidity non condensing
Communication	Bluetooth Smart
Battery	AAA size Alkaline battery 1.5 V
Battery life	ON: 25 - 35 hours OFF: 3 years (shelf life with battery mounted)
Battery Low warning:	@ 1.1 V, typically 4 hours left
Unit turns off:	@ 0.95 V nominal
Shock detection:	5 – 360 J, +/- 15%

Warranty

Refer to the Laerdal Global Warranty for terms and conditions.

For more information visit www.laerdal.com

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trademarks owned by Laerdal Medical AS.
© 2025 Laerdal Medical AS. All rights reserved.