EN

FR

DE

ES

IT

PT

NL

DA

NO

SV

FI

PL

JA

KO

HR


ET

RO



SL

Laerdal Silicone Masks

User Guide
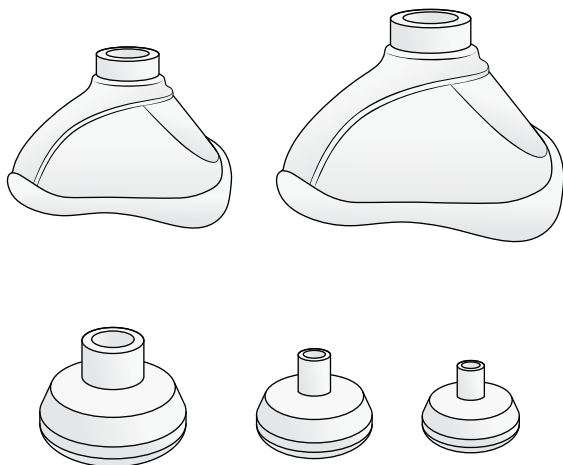


EN	7
FR	13
DE	19
ES	25
IT	31
PT	37
NL	43
DA	49
NO	55
SV	61
FI	67
PL	73
JA	79
KO	85
HR	91
ET	97
RO	103
SL	109

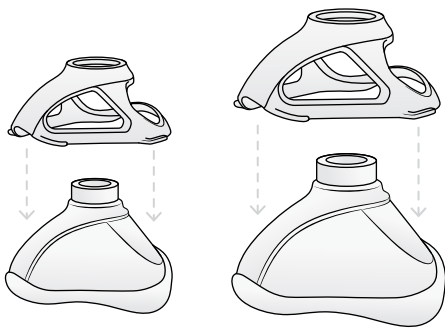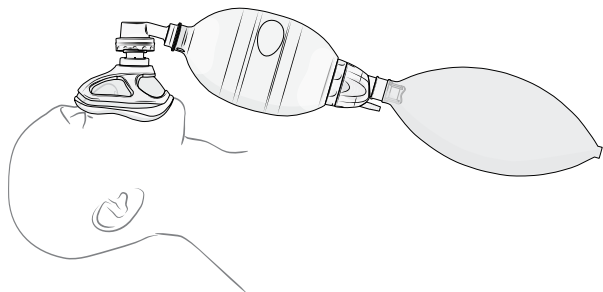
#3-4


#4-5+

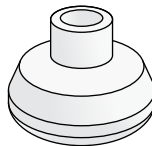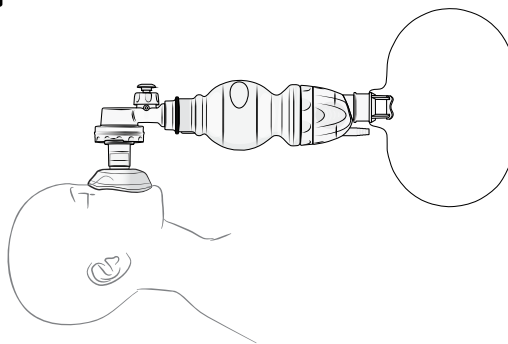#2
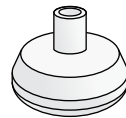
#0/1
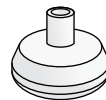
#00

Device Description

Laerdal Silicone Masks (LSM) is a mask that can be connected to a resuscitator.

Indication for Use

Laerdal Silicone Masks (LSM) are indicated for use for patients requiring ventilation.

Intended Use

Laerdal Silicone Masks (LSM) are intended to be used for ventilation of patients.

The Laerdal Silicone Masks are available in five sizes:

- The Adult Silicone Mask size 4-5+ is suitable for adult patients.
- The Child Silicone Mask size 3-4 is suitable for pediatric patients and adult patients with small faces.
- The LSR Silicone Mask size 2 is suitable for infants and small pediatric patients.
- The LSR Silicone Mask size 0/1 is suitable for newborn and small infant patients.
- The LSR Silicone Mask size 00 is suitable for premature newborn patients.

The Multi-Function Mask Cover is intended to be used to add additional support, when needed, to Laerdal Silicone Masks (size 3-4, 4-5+) and to allow for strapping of Laerdal Silicone Masks to the patient.

Intended Users

The LSM is intended to be used by healthcare professionals trained in delivering ventilatory support and in the use of manual resuscitators.

Clinical Benefits

Positive impact on clinical outcome, by respiratory support that reduces probability of adverse outcomes, such as morbidity and mortality caused by hypoxia.

Clinical Outcome

Desired outcome of ventilation is oxygenation of the patient, often evaluated using SpO₂, EtCO₂, blood gas analysis or other method of analysis.

Known Side Effects

Facial damage due to pressure.

Contraindications

No known contraindications for use.

Read this User Guide and become familiar with the operation of the product prior to use.
Use the product only as described in this User Guide.

Warning and Cautions

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or product damage.

Notes

Important information about the product or its operation.

Warnings

- Care should be taken when using the LSM on patients with severe anomalies or when applying other medical devices which may conflict with the mask as mask leakage may occur. If mask face sealing is not possible to achieve consider using alternative airway device.
- Care should be taken when applying pressure to the mask to avoid facial damage, especially in the case of pediatric patients, infants, pre-terms, patients with severe osteoporosis and geriatric patients.
- Care should be taken when choosing the right size mask for the patient to ensure proper fit. Alternative mask sizes should be readily available.

Cautions

- The hard plastic components of the mask cover are incompatible with polar solvents such as ethanol and isopropyl alcohol
- Do not use rinsing or drying agents as these may damage the materials.

Note

Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

To reduce the risk of cross-contamination, follow these instructions after each use.

Cautions

- The mask is not provided sterile. The mask must be cleaned and disinfected prior to initial use.
- It is recommended that the highest level of disinfection/sterilization possible is used for patients that may have compromised immune defense, such as a pre-term baby or in the case of outbreaks of highly transmissible pathogens.
- The components must be cleaned and disinfected before next patient use.
- The use of cleaning and disinfection procedures not described in this section may have adverse effects on the material and/or performance and may not be effective for disinfecting the Mask.
- The hard plastic components of the mask cover are incompatible with polar solvents such as ethanol and isopropyl alcohol.
- To reduce the risk of cross-contamination, follow these instructions after each use.
- If the mask is stored as back-up in an area with potentially high levels of airborne pathogens, it should be considered to store the mask in an air-tight container to avoid contamination.

Inspection

Carefully inspect all parts for signs of wear or damage. Worn or damaged components must be discarded and replaced with new components.

Disassembly

If mask cover is used, disassemble mask and mask cover before cleaning and disinfecting.

Washing and Rinsing

Masks must be cleaned before high-level disinfection or sterilisation.

Masks can be manually cleaned, or cleaned in an automatic washer/disinfectors.

MANUAL CLEANING

Rinse parts under cold running water.

Submerge parts in water at 30 - 40 °C (86 - 104 °F).
Ensure that all surfaces are submerged for at least 2 minutes.

Submerge all parts in water at 60 - 70 °C (140 - 158 °F) which contains dish washing detergent.

Thoroughly clean all surfaces using a brush as necessary.

Rinse all components in detergent-free water at 30 - 40 °C (86 - 104 °F).

Dry the components thoroughly. Inspect all components to confirm that they are clean and dry. If parts are worn or damaged, discard them.

AUTOMATIC CLEANING (applies to all parts)
Washer/Disinfector
Place parts in wire baskets.
Cycle 1: 90 - 95 °C (194 - 203 °F) for more than 12 seconds.
Total process time: approx. 52 min.
Cycle 2: Use a Non-enzymatic alkaline detergent containing 2 - 5% NaOH.
 Caution Do not use rinsing and drying agents.

To obtain high-level disinfection/sterilisation of the Masks, follow one of these methods.

Sterilisation/High-level Disinfection			
Method	Process Parameters		Post-Treatment
	Temperature / Concentration	Exposure time	
Sterilisation <i>(applies to all parts)</i>			
Steam Autoclaving (prevacuum-pulse)	Autoclave at 134 - 137 °C (273 - 279 °F)	3 min (+30s)	Allow parts to cool and dry
High-level Disinfection <i>(applies to all parts)</i>			
Sodium Hypochlorite	0.5% solution	20 min	Remove traces of disinfectant by rinsing in warm tap water, 30 - 40 °C (86 - 104 °F), for at least 2 mins. Dry the components thoroughly.

Conditions	
Operating Conditions	Temperature: -18 °C to 60 °C (0 °F to 140 °F) Humidity: 15% to 95% rH
Storage Conditions	Temperature: -40 °C to 70 °C (-40 °F to 158 °F) Humidity: 15% to 95% rH
Material Chart	
Mask	Silicone
Cover	PSU
Lifetime Parameters	
Shelf-life	5 years
Expected Service Life	100 cycles of reprocessing

Masks - Main Products

Catalogue #	Description
851500xx	LSR Silicone Mask no. 00
851600xx	LSR Silicone Mask no. 0/1
851700xx	LSR Silicone Mask no. 2
860220xx	Child Silicone Mask 3-4 with Multi Function Mask Cover
860221	Child Silicone Mask 3-4 without Multi Function Mask Cover
870220xx	Adult Silicone Mask 4-5+ with Multi Function Mask Cover
870221	Adult Silicone Mask 4-5+ without Multi Function Mask Cover
872220	Adult & Child Silicone Mask with Multi Function Mask Covers

Note

Catalog numbers ending with xx denotes local language configurations

Masks - Spare Parts/Accessories

Catalogue #	Description
865200	Multi Function Mask Cover for Mask 3-4
875200	Multi Function Mask Cover for Mask 4-5+

For latest version of Spare Parts and Accessories, visit www.laerdal.com

Symbol Glossary	
	Medical Device
	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.
	Caution: Federal law restricts this device to sale by, or on the order of a physician (US).
	Not made with natural rubber latex

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© 2025 Laerdal Medical AS. All rights reserved.